

Wheelchair Platform Scale KERN MWA



Wheelchair platform scale with two integrated access ramps for easy access – with approval for professional medical use in medical diagnostics, verification optional



KERN MWA 300K-1PM



KERN MWA 300K-1M



▲ KERN MWA 300K-1M + MWA-A04

With the two practical handrails of the integrated folding patient seat, you can easily convert the wheelchair platform scale into a multi-functional scale. This way patients can be weighed standing up, with support from the handrail or in a comfortable seated position. If you fold the patient seat up, then a wheelchair can be pushed or driven onto the scale. The strut rod included with delivery ensures a high level of stability and torsional rigidity. The drive-through protection which is also fitted as standard for the front wheels of the wheelchair gives an extra level of safety

Wheelchair Platform Scale KERN MWA



Features

- Verification class III (verification is optional)
- Approved as a medical device according to 93/42/EEC or regulation (EU) 2017/745
- Especially suitable for weighing patients in wheelchairs because of the low-profile platform which can be approached from either side
- **1** The large steel weighing plate also makes it ideal for weighing obese patients
- Level indicator to level the balance precisely
- Hold function: While weighing patients that are unable to sit or stand still, a mean average weight value is determined. This allows for sufficient time to attend to the patient, and then get a weight reading
- KERN MWA 300K-1PM: Wheelchair platform scale with practical handrail incl. column for easy, ergonomic mounting of the display device. The handrail offers patients that extra level of safety when driving or stepping onto the weighing platform
- **2** Both access ramps and the handrail can be folded up quickly and easily, e.g. for transportation
- BMI function to determine underweight/normal weight/surplus weight
- Battery- or mains-powered, rechargeable battery operation optional
- High resolution readability: readability [d] can be increased by one decimal place for 5 sec. by the touch of a key

- GLP/ISO report, printout of weight value, BMI etc. with date and time. FLP/GMP-compliant print form in DE, EN, FR, only in combination with RS-232 interface cable KERN CFS-A01 and printer KERN YKB-01N
- Protective working cover included with delivery

Technical data

- Large backlit LCD display, digit height 25 mm
- Dimensions weighing surface WxD 880x840 mm
- Dimensions of display device WxDxH 200x130x60 mm
- Cable length of display device approx. 3 m
- Overall dimensions, without handrail, WxDxH 1180x1130x90 mm
- Battery operation possible, 6x1.5 V AA not included, operating time up to 20 h
- Mains adapter external, standard
- Net weight approx. 70 kg

3 ◀ **KERN MWA-A02:** Handrail, without column, cannot be folded, for self-assembly on the wheelchair platform scale KERN MWA 300K-1M or MWA 300K-1PM. With the practical optional handrail set, the wheelchair platform scale KERN can be converted quickly and easily into a multi-function scale. This allows the patients to be weighed while standing on the scale, with support from the handrail or weighed in a wheelchair

Accessories

- Protective working cover over the display device, scope of delivery: 5 items, KERN MBC-A06S05
- Cleaning cloths, alcohol-free cloths for disinfectant wiping, quick acting, based on modern quaternary ammonium compounds and effective against papova viruses. Particularly gentle on materials, and very well suited for disinfecting products which are sensitive to alcohol. Fulfill the legal requirements for occupational safety in accordance with TRGS 525/540. Packaging contents 100 pcs., size 20x22 cm per cloth, KERN MYC-01
- Internal rechargeable battery pack, operating time up to 40 h without backlight, charging time approx. 12 h, KERN MBC-A08
- External mains adapter, 100 V - 240 V, Standard EU, CH, UK, KERN YKA-43
- External mains adapter, 100 V - 240 V, Standard EU, KERN YKA-44
- Interface cable RS-232 to connect an external device, KERN CFS-A01
- **3** Handrail, lacquered steel, dimensions WxDxH approx. 560x40x920 mm, scope of supply: handrail, fasteners, net weight approx. 4,0 kg, KERN MWA-A02
- **4** MWA 300K-1M: Handrail kit with folding patient seat, please add this to your order, retrofitting is not possible, KERN MWA-A04
- Matrix needle printer, KERN YKN-01
- Affordable universal label printer to print out weights on thermal labels. ASCII-capable. Convenient Auto-Forward function to automatically jump from label to label (only with KERN standard labels), KERN YKE-01
- Thermal printer, KERN YKB-01N
- For further details, plenty of further accessories and suitable printers, see Internet

*Within the EU, official verification (conformity assessment according to NAWI 2014/31/EU) is mandatory by law for scales that are intended for use as a medical device. Please add this to your order. We require the location of use and the post code for the verification

STANDARD



OPTION



FACTORY



0297

Model	Weighing range	Readout	Verification value	Mandatory by law
	[Max] kg	[d] kg	[e] kg	Verification
KERN				KERN
MWA 300K-1M	300	0,1	0,1	965-129
MWA 300K-1PM	300	0,1	0,1	965-129

CAL EXT
Adjusting program CAL
 For quick setting up of the balance's accuracy. External adjusting weight required

MEMORY
Memory
 Balance memory capacity, e.g. for article data, weighing data, tare weights, PLU etc.

RS 232
Data interface RS-232
 To connect the balance to a printer, PC or network

RS 485
RS-485 data interface
 To connect the balance to a printer, PC or other peripherals. Suitable for data transfer over large distances. Network in bus topology is possible

USB
USB data interface
 To connect the balance to a printer, PC or other peripherals

BT
Bluetooth* data interface
 To transfer data from the balance to a printer, PC or other peripherals

WIFI
WIFI data interface
 To transfer data from the balance to a printer, PC or other peripherals

SWITCH
Control outputs (optocoupler, digital I/O)
 To connect relays, signal lamps, valves, etc.

STATISTIC
Statistics
 Using the saved values, the device calculates statistical data, such as average value, standard deviation etc.

SOFTWARE
PC Software
 to transfer the measurements from the device to a PC

GLP INTERN
GLP/ISO log internal
 The balance displays weight, date and time, independent

GLP PRINTER
GLP/ISO log
 With date and time. Only with KERN printers

KCP PROTOCOL
KERN Communication Protocol (KCP)
 It is a standardized interface command set for KERN balances and other instruments, which allows retrieving and controlling all relevant parameters and functions of the device. KERN devices featuring KCP are thus easily integrated with computers, industrial controllers and other digital systems

PCS
Piece counting
 Reference quantities selectable. Display can be switched from piece to weight

SUM
Totalising level A
 The weights of similar items can be added together and the total can be printed out

UNIT
Weighing units
 Can be switched to e.g. nonmetric units. Please refer to website for more details

TOL
Weighing with tolerance range (Check weighing)
 Upper and lower limiting can be programmed individually, e.g. for sorting and dosing. The process is supported by an audible or visual signal, see the relevant model

ZERO
ZERO
 Resets the display to "0"

MOVE
Hold function
 When patients do not stand, sit or lie completely still, a stable weight is calculated using an average weight

MOVE
Hold function
 When the weighing conditions are unstable, a stable weight is calculated as an average value

IP
Protection against dust and water splashes IPxx
 The type of protection is shown in the pictogram of. DIN EN 60529:2000-09, IEC0529:1989+A1:1999 +A2:2013

UNDER
Suspended weighing
 Load support with hook on the underside of the balance

BATT
Battery operation
 Ready for battery operation. The battery type is specified for each device

RECHARGE
Battery operation rechargeable
 Prepared for a rechargeable battery operation

ACCU
Rechargeable battery pack
 Rechargeable set

MULTI
Universal plug-in power supply
 with universal input and optional input socket adapters for
 A) EU, CH
 B) EU, CH, GB, US
 C) EU, CH, GB, US, AUS

230 V
Plug-in power supply
 230V/50Hz in standard version for EU. On request GB, AUS or US version available

230 V
Integrated power supply unit
 Integrated in balance. 230V/50Hz standard EU. More standards e.g. GB, AUS or US on request

DMS
Weighing principle Strain gauges
 Electrical resistor on an elastic deforming body

PEAK
Peak hold function
 capturing a peak value within a measuring process

PUSH/PULL
Push and Pull
 the measuring device can capture tension and compression forces

SCALE
Integrated scale
 In the eyepiece

360°
360° rotatable microscope head

360°
Monocular Microscope
 For the inspection with one eye

BINDO
Binocular Microscope
 For the inspection with both eyes

TRINO
Trinocular Microscope
 For the inspection with both eyes and the additional option for the connection of a camera

ABBE
Abbe Condenser
 With high numerical aperture for the concentration and the focusing of light

HAL
Halogen illumination
 For pictures bright and rich in contrast

LED
LED illumination
 Cold, energy-saving and especially long-life illumination

FL-HBD
Fluorescence illumination for compound microscopes
 With 100 W mercury lamp and filter

FL-LED
Fluorescence illumination for compound microscopes
 With 3 W LED illumination and filter

PH
Phase contrast unit
 For a higher contrast

DF
Darkfield condenser/unit
 For a higher contrast due to indirect illumination

POLAR
Polarising unit
 To polarise the light

INFINITY
Infinity system
 Infinity corrected optical system

AUTO ATC
Automatic temperature compensation
 For measurements between 10 °C and 30 °C

M +3 DAYS
Conformity assessment
 The time required for conformity assessment is specified in the pictogram

1 DAY
Package shipment
 The time required for internal shipping preparations is shown in days in the pictogram

2 DAYS
Pallet shipment
 The time required for internal shipping preparations is shown in days in the pictogram

*The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by KERN & SOHN GmbH is under license. Other trademarks and trade names are those of their respective owners.